

iTero® Element 5D Wand Barrier Sleeve Instructions For Use


WARNING: To prevent cross contamination, it is essential that the wand and cradle are fully cleaned and disinfected after each patient, as described in the *iTero Element 5D User Manual*.

WARNING: For single use only. Do not re-use. To prevent cross contamination, it is essential to replace the wand barrier sleeve after each patient.


CAUTION: Dispose of wand barrier sleeves according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Applying a new wand barrier sleeve


- 1) Place a new wand barrier sleeve in its packaging near the iTero Element 5D wand.
- 2) Gently remove the protective sleeve from the wand.

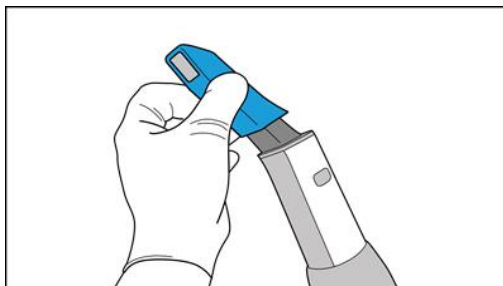


- 3) Visually inspect the sleeve packaging to verify that there is no damage. Replace it if any damage is observed.
- 4) Wearing gloves, remove the wand barrier sleeve from its packaging, making sure to keep the transparent sheath folded.

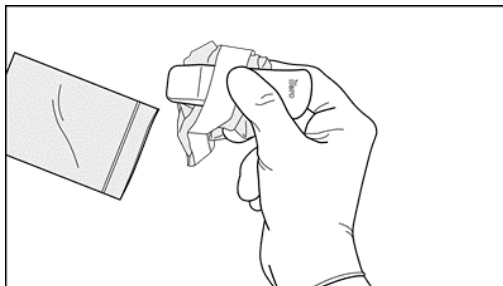

- 5) Slide the rigid section of the wand barrier sleeve onto the optical tip of the wand until you feel a "click", making sure that the transparent sheath is not pinched between the rigid section of the wand barrier sleeve and the wand.

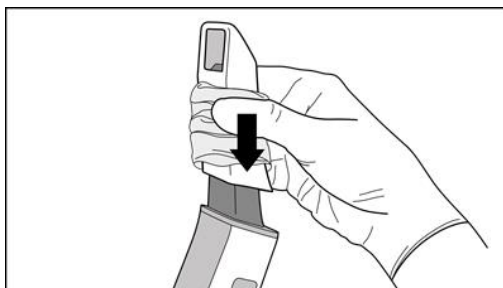

- 6) Gently pull the transparent sheath to cover the entire wand.

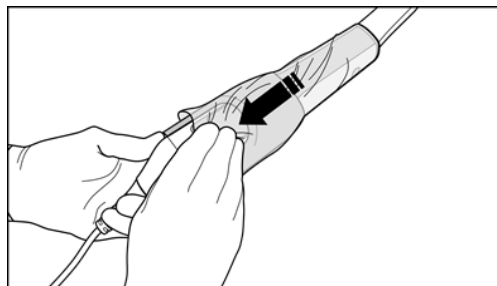


Note: The white tape on the wand barrier sleeve is for manufacturing purposes only, and should be ignored.


- 7) Visually inspect the wand barrier sleeve for any noticeable damage, for example, cracks, scratches, and holes. Replace it in case of noticeable damage.

Removing the wand barrier sleeve


- 1) Wearing gloves, scrunch the transparent sheath half way toward the wand tip.

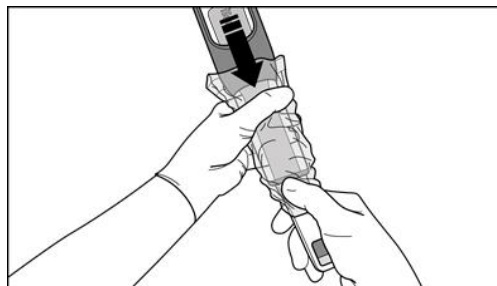

- 2) Holding the wand by the scrunched transparent sheath, slightly release the rigid section of the barrier sleeve from the wand tip using the other hand.

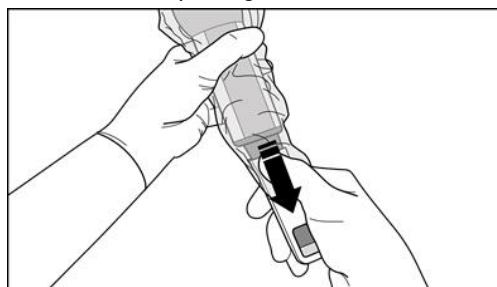

- 3) Holding the wand by the scrunched transparent sheath, place the wand in the cradle.
- 4) Pull the rigid section of the wand barrier sleeve to remove it from the wand.

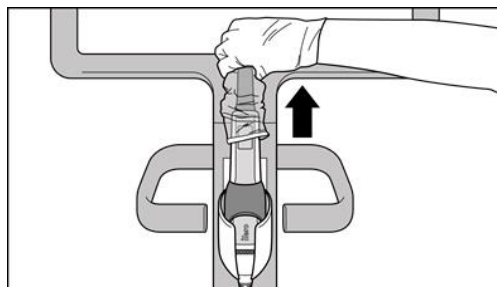

Note: Do not touch the optical surface of the wand, as this may cause damage.

