Addendum to the iTero™ scanners user manuals

Align Technology Ltd.
1 Yitzhak Rabin Rd.,
Petach Tikva, 4925110, Israel

Align Technology B.V.
Herikerbergweg 312
1101 CT, Amsterdam The Netherlands

Last updated November 2025

1 Purpose of this document

This document complements the current iTero user manuals and provides information regarding the new Gumlines tool in Align Oral Health Suite.

2 Working with the Gumlines tool

The Gumlines tool in Align Oral Health Suite is intended to visualize the position of the gum line and the cemento-enamel-junction (CEJ) on a patient-specific 3D model of the oral cavity. The software application is indicated to enhance communication between the patient and the dental healthcare providers, for education purposes.

The tool displays two lines on the model – a solid black line that depicts the patient's current gum line, and a dotted purple line that shows the patient's CEJ.

The Gumlines tool does not treat or diagnose any disease.

Disclaimers and limitations:

- The Gumlines visual should be used at the clinician's own discretion.
- The CEJ may be inaccurately visualized if it resides under the gum line, along with other factors:
 - Partial scan capture, holes in scan
 - Ongoing orthodontic braces treatment or Invisalign buttons
 - Severe gingivitis and hyperplasia
 - Missing teeth (no CEJ) and uninterrupted dentition (CEJ is subgingival)
 - Distal molars
- This software is not intended for use:
 - As a virtual anatomical model for (1) the diagnosis or monitoring of a disease, injury or disability, or (2) the investigation of the anatomy or of a physiological process
 - To provide therapy to a person through the provision of information
 - To recommend a treatment or intervention to a Healthcare Professional or patient
 - For monitoring the state or progression of a disease or condition
 - In relation to diagnosing or screening for a disease or condition
- The tool does not alter, modify, or corrupt the original data generated by the iTero scanner.

To display the gum and CEJ lines on the model:

1. In Align Oral Health Suite, tap **Gum Health** and then tap the Gumlines button .

The **AI Insights** pane is displayed on the screen, and the gum line (solid black line) and the CEJ line (dotted purple line) are displayed on the 3D model.

Figure 1: Gum line and CEJ line displayed on the model

2. In the **AI Insights** pane, tap the relevant visibility toggles  or  to hide or show the AI Insights, the gum line, and/or the CEJ.

Português do Brasil (Brazilian Portuguese)	5
Български (Bulgarian).....	8
Hrvatski (Croatian).....	11
Čeština (Czech).....	14
Dansk (Danish).....	17
Nederlands (Dutch)	20
Eesti keel (Estonian).....	23
Suomi (Finnish)	26
Français (French)	29
Deutsch (German).....	32
Ελληνικά (Greek).....	35
עברית (Hebrew)	38
Magyar (Hungarian).....	41
Italiano (Italian)	44
日本語 (Japanese).....	47
Қазақша (Kazakh).....	50
한국어 (Korean).....	53
Latviešu (Latvian)	56
Lietuvių (Lithuanian)	59
Norsk (Norwegian).....	62
Polski (Polish).....	65
Português (Portuguese)	68
Română (Romanian)	71
Русский (Russian).....	74
Srpski (Serbian).....	77
简体中文 (Simplified Chinese).....	79
Slovenčina (Slovak).....	82
Slovenščina (Slovenian).....	85
Español (Spanish)	88
Svenska (Swedish).....	91
繁體中文 (Traditional Chinese).....	94
Türkçe (Turkish)	97
Українська (Ukrainian).....	100
Tiếng Việt (Vietnamese).....	103

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© 2025 Align Technology, Inc. All rights reserved. Align, Invisalign, iTero, iTero Lumina, among others, are trademarks and/or service marks of Align Technology, Inc. or one of its subsidiaries or affiliated companies and may be registered in the U.S. and/or other countries. 227196 Rev. B
Align Technology, Inc.